www.annecy-lentilles.fr

Deutsch

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www.annecy-lentilles.fr

Information for Contact Lens Wearers

Contact lens care system CONTOPHARMA Conventional

CONTOPHARMA cleaning solution: Cleaning solution with optimal very efficient surface-active agents, suitable for the cleaning of all current types of rigid gas-permeable as well as soft hydrophilic contact lenses (50 ml).

CONTOPHARMA conservation and rinsing: Solution for conservation and rinsing of all current types of rigid gas-permeable as well as soft hydrophilic contact lenses (50 – 250 ml).

CONTOPHARMA saline solution: For rinsing and moisturizing of all current types of rigid gas-permeable as well as soft hydrophilic contact lenses, can be used for thermal disinfection as well (250 ml).

Warnings:

To avoid complications with the system/eyes/contact lenses, contact lens care products should be used only after consulting with a contact lens specialist, taking into consideration the lens material used. Wearing contact lenses that have not been cleaned properly or are damaged can lead to serious injury to the eyes or visual impairment. If a reddening of the eye occurs on the contact lens wearing surface or the conjunctiva, one should consult an ophthalmologist or a contact lens specialist immediately.

Special Instructions for use:

- Avoid any contact of *CONTOPHARMA cleaning solution* with your eyes
- If the cleaning solution should get into your eyes, flush your eyes with saline solution and consult your contact lens specialist and/or an ophthalmologist if any irritation persists.
- Rinse the contact lenses only with *CONTOPHARMA conservation and rinsing solution* or *CONTOPHARMA saline solution*, never with tap water.
- Once the bottle has been opened, the solution may not be used for more than 60 days (make a note of the first use on the bottle). Any remaining contents after 60 days should be properly disposed of.
- Do not use a bottle if the security seal has been damaged.
- Store the solution at room temperature (15–25°C) and protect from direct light.
- Observe general hygiene measures such as hand washing before lens care.
- To prevent possible microbial contamination of the solution, do not touch the tip of the bottle. Close the bottle immediately after using and keep tightly closed.
- Keep out of the reach of children.
- The *CONTOPHARMA cleaning solution* should not be used for storing contact lenses!
- For care and storage of your contact lenses, use care solutions and storage containers recommended by your contact lens fitter.

Instructions for use:

Please observe the following instructions carefully:

1. Wash hands thoroughly and dry with a clean towel. Do not use re-greasing soaps for the hand washing.
2. Place several drops (approx. 2–4) of *CONTOPHARMA cleaning solution* on both sides of the contact lens.
3. **Rigid gas-permeable contact lenses:** Rub between the index finger, middle finger and the thumb for 20–30 seconds.
Soft hydrophilic contact lenses: Clean both sides in the palm of one hand with the aid of the index finger of the other hand for 20–30 seconds.
4. Thoroughly rinse off any dissolved dirt particles and the cleaning solution with the *CONTOPHARMA conservation and rinsing solution* or *CONTOPHARMA saline solution*.
5. Close the bottle immediately after use.
6. Store the cleaned contact lenses overnight in a contact lens storage case filled with the *CONTOPHARMA solution for conservation and rinsing*.
7. Before inserting the contact lenses, rinse once again with the *CONTOPHARMA conservation and rinsing solution* or with *CONTOPHARMA sterile saline solution*.
8. Discard the residue and rinse the safety case well under running water and have the case dried well open.

If the lenses are not worn for several weeks, they may be stored in the closed *CONTOPHARMA safety case*. Before a renewed use, steps 1–5 should be repeated as described.

To date no sideeffects for the *CONTOPHARMA care system Conventional* have been observed when used according to instructions.

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Laboratorium Dr. G. Bichsel AG
Contopharma AG,